## ENGLISH

1. Trademark  
2. Reference  
3. CE marking : PPE Category III

4. Size and body measurements  
5. Refer to instructions for use  
6. Fabric care symbols

### PERFORMANCE PHYSIQUE OF WEECOVER® ECO

Fabric Physical Properties	Test Method	Class
Abrasion resistance	EN 530	2
Flex cracking resistance	ISO 7854 B	6
Trapezoidal Tear resistance	ISO 9073-4	2
Tensile strength	ISO 13934-1	2
Puncture resistance	EN 863	1
Seam strength	ISO 13935-2	3

### Fabric resistance to penetration by liquids EN ISO 6530

Fabric Chemical Properties	Penetration	Repellency
Sulphuric Acid 30%	Class 3	Class 3
Sodium Hydroxide 10%	Class 3	Class 3
Butan-1-ol	Unclassified	Unclassified
o-xylene	Unclassified	Unclassified

### Whole suit test performance

Inward leakage test as defined by EN ISO 13982-1:2004 + A1:2010	Results
Light splashes Method as defined by EN 13034 : 2005 +A1: 2009	Pass

### Typical Area of use

White coveralls with elasticated hood, ankles and back, knitted cuffs, adhesive flap over the zipper, thumb loop (optional). Coveralls are designed to protect against chemical risks (splashes – light sprays - airborne solid particulates).

### How to wear protective clothing

Remove the coverall from its packaging, open the center zipper and put on the garment, taking care not to damage it. Close the zipper completely and cover it with the adhesive flap, without creasing. If there is a risk of airborne solid particles, it is advisable to tape additionally protective gloves with ends of the sleeves.

### Limitations of use

Do not use the coverall if any defects are noticed (e.g.: holes, damaged seams, ...). The coverall meets the requirements: Lijnm 82/90 ≤ 30% - Ls 8/10 ≤ 15%. Type 6 chemical protective suits have been tested to the whole suit test. Wear for long periods of time can cause heat stress. Heat stress and discomfort can be reduced or eliminated by using appropriate undergarments or suitable ventilation equipment. Coveralls are for single use only and must be disposed after any job. The user is the sole judge of the adequacy between his risk analysis and the type of protection required, including the additional equipment necessary for a complete protection of the body. If necessary, use additional PPE with the same level of protection (such as gloves, breathing apparatus, etc.) in order to provide full body protection. The manufacturer cannot accept responsibility for any improper use or disposal of garments produced by them. These garments are flammable, keep away from fire. Use the product within 5 years of the date of manufacturing.

### Storage and disposal

Coveralls should be stored in the original packaging in a dry place, away from sunlight. They must be disposed of in compliance with current regulations.

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