## ENGLISH

- Brand name
- Model identification
- CE marking : PPE Category III
- Size and body measurements
- Refer to instructions for use
- Fabric care symbols

PERFORMANCE PHYSIQUE OF WEECOVER® ECO		
<b>Fabric Physical Properties</b>	<b>Test Method</b>	<b>Classe</b>
Abrasion resistance	EN 530	2
Flex cracking	ISO 7854 B	6
Trapezoidal Tear	ISO 9073-4	2
Tensile strength	ISO 13934-1	2
Puncture resistance	EN 863	1
Seam strength	ISO 13935-2	3

Fabric resistance to penetration by liquids EN ISO 6530		
<b>Fabric Chemical Properties</b>	<b>Penetration</b>	<b>Repellency</b>
Sulphuric Acid 30%	Class 3	Class 3
Sodium Hydroxide 10%	Class 3	Class 3
Whole suit test performance		<b>Results</b>
Dry particles Method as defined by EN ISO 13982-1:2004 + A1:2010		Pass
Light splashes Method as defined by EN 13034 <span> </span> : 2005 +A1: 2009		Pass
Surface resistivity EN1149-5 <span> </span> : 2018		Pass

Typical Area of use
White coveralls with elasticated hood, ankles and back, knitted cuffs, adhesive flap over the zipper, thumb loop (optional). Coveralls are designed to protect against chemical risks (splashes – light sprays - airborne solid particulates) and electrostatics (dissipative properties).

How to wear protective clothing
Remove the coverall from its packaging, open the center zipper and put on the garment, taking care not to damage it. Close the zipper completely and cover it with the adhesive flap, without creasing. If there is a risk of airborne solid particles, it is advisable to tape additionally protective gloves with ends of the sleeves.

Limitations of use
Do not use the overall if any defects are noticed (e.g.: holes, damaged seams, ...). The overall meets the requirements: L <sub>imn</sub> 82/90 ≤30% - L <sub>s</sub> 8/10 ≤ 15%. Type 6 chemical protective suits have been tested to the whole suit test. Wear for long periods of time can cause heat stress. Heat stress and discomfort can be reduced or eliminated by using appropriate undergarments or suitable ventilation equipment. Coveralls are for single use only and must be disposed after any job. The user is the sole judge of the adequacy between his risk analysis and the type of protection required, including the additional equipment necessary for a complete protection of the body. If necessary, use additional PPE with the same level of protection (such as gloves, breathing apparatus, etc.) in order to provide full body protection. The manufacturer cannot accept responsibility for any improper use or disposal of garments produced by them. These garments are flammable, keep away from fire. The person wearing the electrostatic dissipative protective clothing shall be properly earthed. The resistance between the person skin and earth shall be less than 10 <sup>8</sup> Ω e.g. by wearing adequate footwear on dissipative or conductive floors. Electrostatic dissipative protective clothing shall not be open or removed whilst in presence of flammable or explosive atmospheres or white handling flammable or explosive substances. Electrostatic dissipative protective clothing is intended to be worn in Zones 1, 2, 20, 21 and 22 (see EN 60079-10-1 and EN 60079-10-2) in which the minimum ignition energy of any explosive atmosphere is not less than 0,016 mJ. Electrostatic dissipative protective clothing shall not be used in oxygen enriched atmospheres, or in Zone 0 (see EN 60079-10- 1) without prior approval of the responsible safety engineer. The electrostatic dissipative performance of the electrostatic dissipative protective clothing can be affected by wear and tear, laundering and possible contamination electrostatic dissipative protective clothing shall be worn in such a way that it permanently covers all non-complying materials during normal use. Use the product within 5 years of the date of manufacturing.

Storage and disposal
The coveralls must be stored in their original packaging in a dark, dray place. They must be disposed of in compliance with current regulations.

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